


AccuTwist

INSTRUCTIONS FOR FILLING

Read all instructions prior to filling.

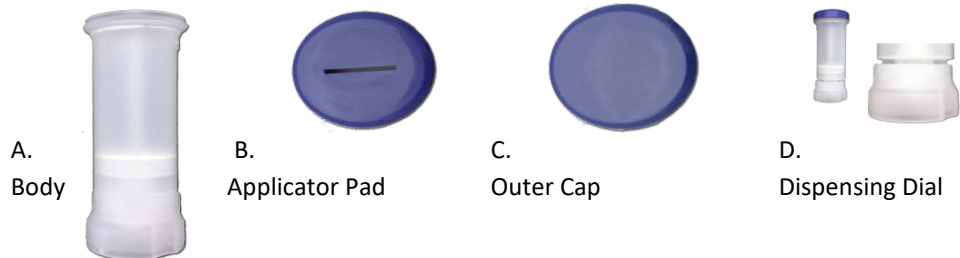

Filling Instructions


1. Fill the Accu-Twist Body with topical product. Periodically tap the Accu-Twist on the counter top to remove air pockets.



2. Fill past the top with product as required. There is area in the cap for extra cream.



3. **NOTE: Do Not Separate the Outer Cap from the Applicator Pad.** Place the Cap and Pad on the top of the Body. Using both hands, press down together to snap the Pad onto the Body.

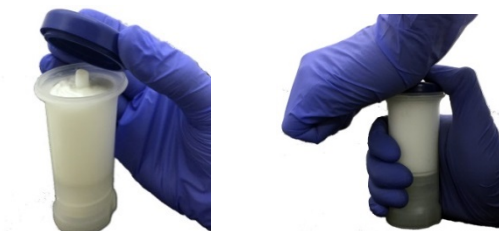

4. Prime the topical by first removing the Outer Cap, then pulling down the Dispensing Dial and rotating counter clockwise until the topical dispenses from the top slot. Continue to rotate until the dial locks into the body. Wipe off the topical from the Applicator Pad.



5. Be sure the Applicator Pad is snapped on Body.



6. Press the Outer Cap onto Applicator Pad and apply the label to the Accu-Twist dispenser. **Note: Do not apply label on top of the solid line on the side of the dispenser.**

