


# AccuTwist

## INSTRUCTIONS FOR USAGE

**Read all instructions prior to using.**

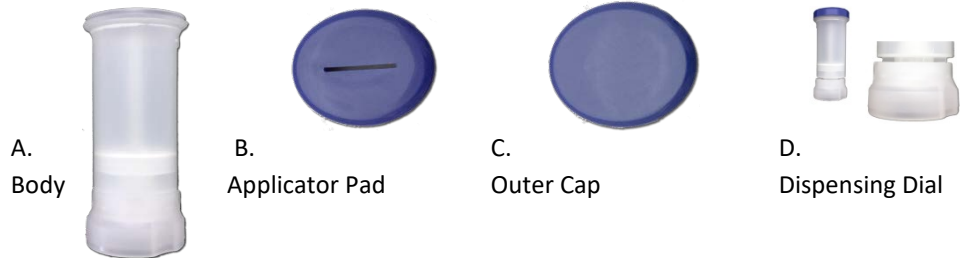


### Usage Instructions

1. Remove the Outer Cap from the top of the Accu-Twist dispenser.



2. Pull down the Dispensing Dial located at the bottom of the dispenser. Rotate the dial counter clockwise until the topical dispenses from the top slot. Release the dial, but continue to rotate until the dial locks into the body. Repeat as many turns as instructed before applying.

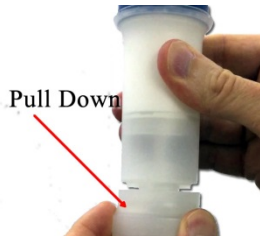

3. Check to make sure cream dispensed onto the applicator pad.



4. Apply by following your physician or pharmacist instructions, typically by rubbing the top of the dispenser where the cream is located directly against the skin.



5. Once applied, wipe off any cream on the top. Replace Outer Cap on the top of the dispenser and press down to lock in place. This will prevent evaporation of the topical product.



**BE SURE TO STORE AWAY FROM EXCESSIVE HEAT OR SUNLIGHT**



For further questions and comments, call us at 407-622-5300.

Accu-Twist.com | 407-622-5300 | sales@accu-twist.com

ACCU-TWIST DISPENSERS

*NOTE: If disassembled, small parts can become a choking hazard.*